

► This document is designed as a reference to be used alongside the Xprecia Stride™ Coagulation Analyzer. For complete instructions, see the *Xprecia Stride User Guide* and the instructions for use that came with your test strips and controls.

### PACKAGE CONTENTS


- 1 Xprecia Stride
- 2 Protective Caps (3)
- 3 USB Cable
- 4 AA Batteries (3)

### CONSUMABLES


- 1 PT/INR Test Strips
- 2 PT Control 1
- 3 PT Control 2
- 4 CaCl<sub>2</sub> Diluent

## GETTING STARTED

1 Install batteries.





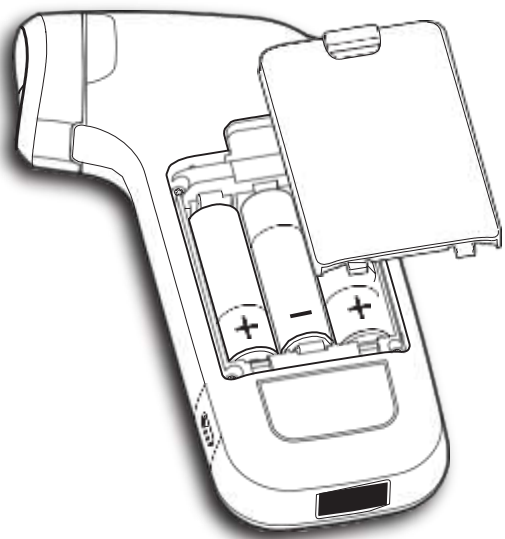


2 Turn Stride on by pressing the On/Off button.

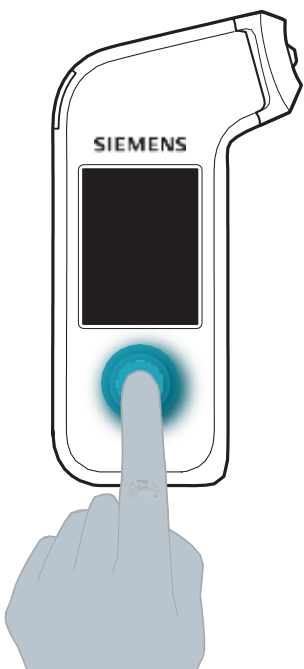

## ANALYZER OVERVIEW

- 1 Test Strip Eject Button
- 2 Battery Power
- 3 Time and Date
- 4 Patient Test
- 5 Settings
- 6 On/Off Button
- 7 Barcode Reader
- 8 Test Strip Port
- 9 Current Screen
- 10 Recall Results
- 11 Quality Control Test
- 12 USB Connection

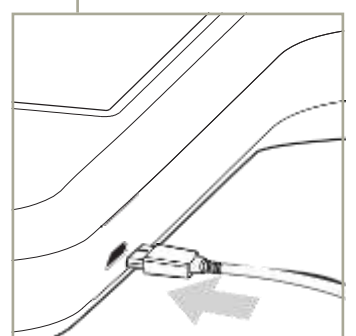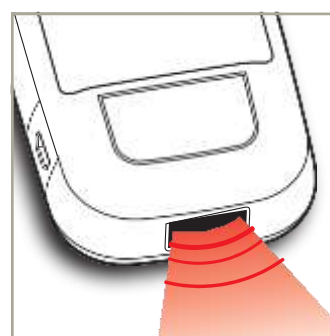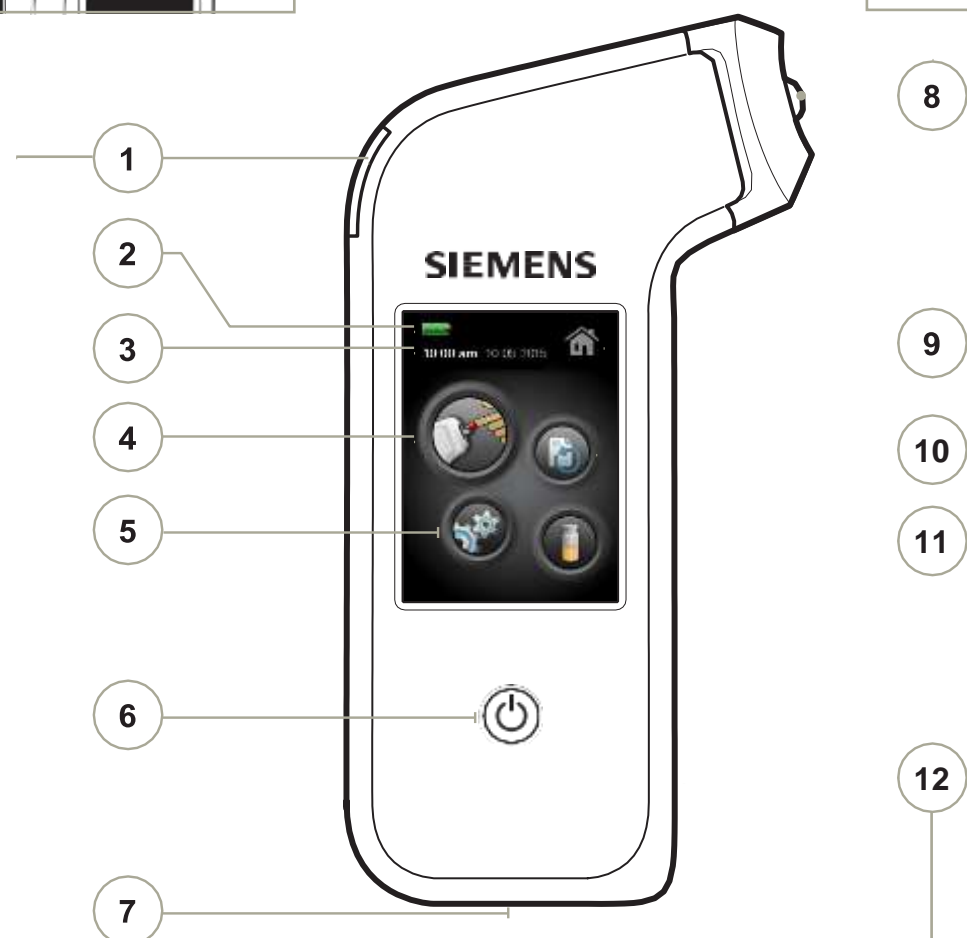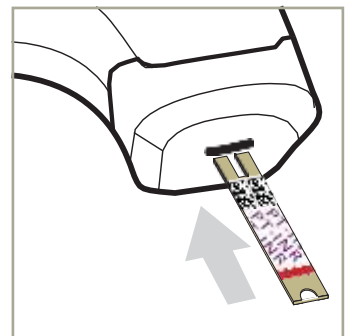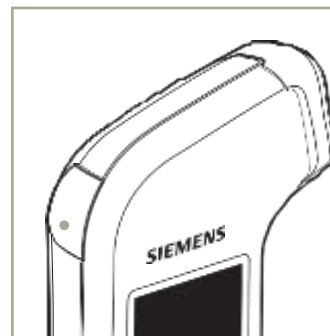

► This document is designed as a reference to be used alongside the Xprecia Stride Coagulation Analyzer. For complete instructions, see the *Xprecia Stride User Guide* and the instructions for use that came with your test strips and controls.

## Patient Test

- 1 Scan Patient ID (optional).
- 2 Scan strip.
- 3 Insert strip.
- 4 Scan strip vial (optional).
- 5 Prepare sample.
- 6 Hold level.
- 7 Apply sample.
- 8 Test in progress.
- 9 View result.
- 10 Eject strip.
- 11 Clean Stride.

## Quality Control Test

For complete cleaning instructions, see the *Xprecia Stride User Guide*.

**IMPORTANT**  
 ◀ Prepare Xprecia Systems PT Controls in advance and allow to stand for 5 minutes. Use the reconstituted control solution within 25 minutes, when stored at 15-25°C (59-77°F). For complete instructions, see the *Xprecia Stride User Guide* and the instructions for use that came with your controls.

- 1 Scan control solution bottle.
- 2 Scan strip.
- 3 Insert strip.
- 4 Scan strip vial (optional).
- 5 Gather controls.
- 6 Hold level.
- 7 Apply control sample.
- 8 Test in progress.
- 9 View result.
- 10 Eject strip.
- 11 Clean Stride.

## REFERENCE INFORMATION

### Settings

#### Settings - Icon Key

	<b>Clock</b> Change the time.		<b>Language</b> Change your language settings.
	<b>Date</b> Change the date.		<b>Information</b> View the software version number.
	<b>Analyzer</b> Customize the volume, brightness, battery type, default settings, and how results are displayed.		<b>Tutorial</b> Learn how to perform a patient test, QC test, or hold Stride.
	<b>Administrator</b> Customize the administrator settings for optional Operator ID (OID), Patient ID (PID), and Login.		

### Recall Results

- 1 Tap a result type.
- 2 Tap a result to view details.
- 3 Tap to view more details.