

# MEDICAL DEVICES

York Teaching Hospital **NHS**  
NHS Foundation Trust

Issue 67  
10th Sept 2020

## Important Information



### Lookout for incorrect T34 Battery

Please be aware that there are some batteries being used within the Trust that should **not** be used with the T34 Ambulatory Syringe Pump.



Information provided from EME:

*These batteries (PROCELL, as pictured) although 6LR61 are significantly smaller both in length and width, even with the pad fitted the terminals are losing contact causing issues with the infusion.*

*If you remove the battery cover on the T34 and turn upside down the battery actually falls out, whereas the NX ones are a tight fit.*

If you do come across the PROCELL within your area then please do not use them on the T34. **The NX is the battery of choice.**

**Please note that purchasing have been made aware of the issue with this particular PROCELL battery**



Further Information can be obtained from:

**Keith Underwood**  
Scarborough

**(771) 2633**

**Alastair Jakeman**  
York

**(772) 6292**