
Routine decontamination of reusable non-invasive patient

care equipment

Routine decontamination of reusable noninvasive care equipment

- Check manufacturer's instructions for suitability of cleaning products especially when dealing with electronic equipment
- Wear appropriate PPE e.g. disposable, non-sterile gloves and aprons

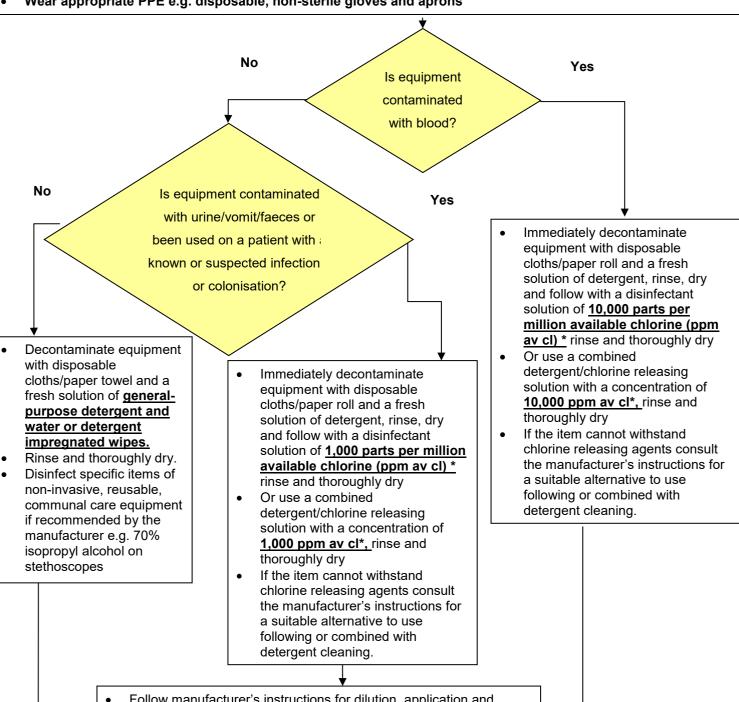

- Follow manufacturer's instructions for dilution, application and contact time.
- Clean the piece of equipment from the top or furthest away point
- Discard disposable cloths/paper roll immediately into the healthcare waste receptacle
- Discard detergent/disinfectant solution in the designated area
- Clean, dry and store re-usable decontamination equipment
- Remove and discard PPE
- Perform hand hygiene

* Scottish National Blood Transfusion service and Scottish Ambulance Service use products different from those stated in the National Infection Prevention and Control Manual